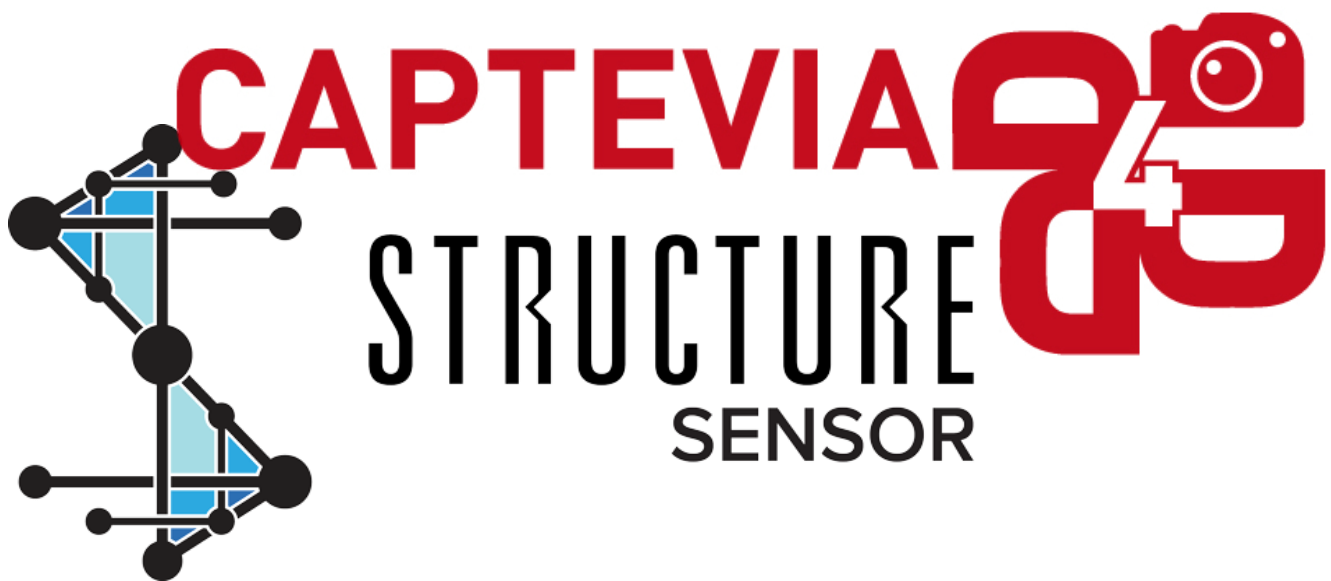
USER GUIDE

Version 3.2

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Version 3. 2

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Versión 3. 2

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Version 3. 2


Summary

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English

What is the structure sensor and Captevia ?

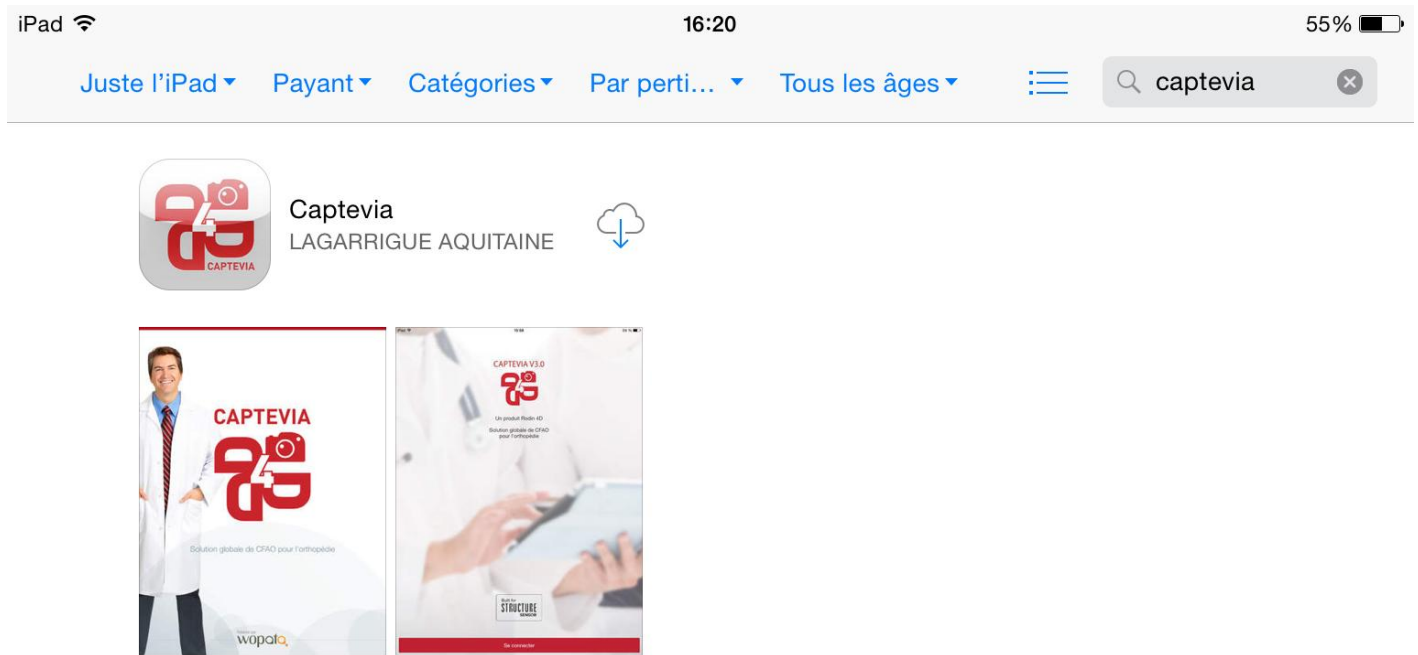Structure Sensor is a 3D scanner that provides you a portable solution for 3D acquisition.
To use it, you must download the application Captevia V3 (or later) from the Apple Store.

Download Captevia V3 on the Apple Store

Compatibility: Requires iOS 7.0 or later.

Subscribe to Captevia

First of all, you must create an account on Captevia. To do so, you must give a valid email address on the first screen of Captevia application.

Your iPad must have an email address configured. If you plan to use dropbox to exchange files, you must download dropbox application on the Apple Store and create an account.

Create a new patient

Create a patient with the “+” button. You must fill in the information and complete the creation.


The screenshot shows the 'Add a patient' form in the Captevia app. The form is overlaid on a 'Patients' list. The form fields include: SURNAME, FIRST NAME, DATE OF BIRTH, HEIGHT (cm), WEIGHT (kg), DOCTOR, APPLICATOR, AGENCY, EQUIPMENT TYPE, and COMMENTS. The 'Add a patient' header has 'Cancel', 'Add a patient', and 'Save' buttons. The background shows a list of patients with names like 'Doe Jane', 'Doe John', 'Kg bu Rage', and 'Rondin Pp'. A keyboard is visible at the bottom.

Create a patient.

Scan your patient

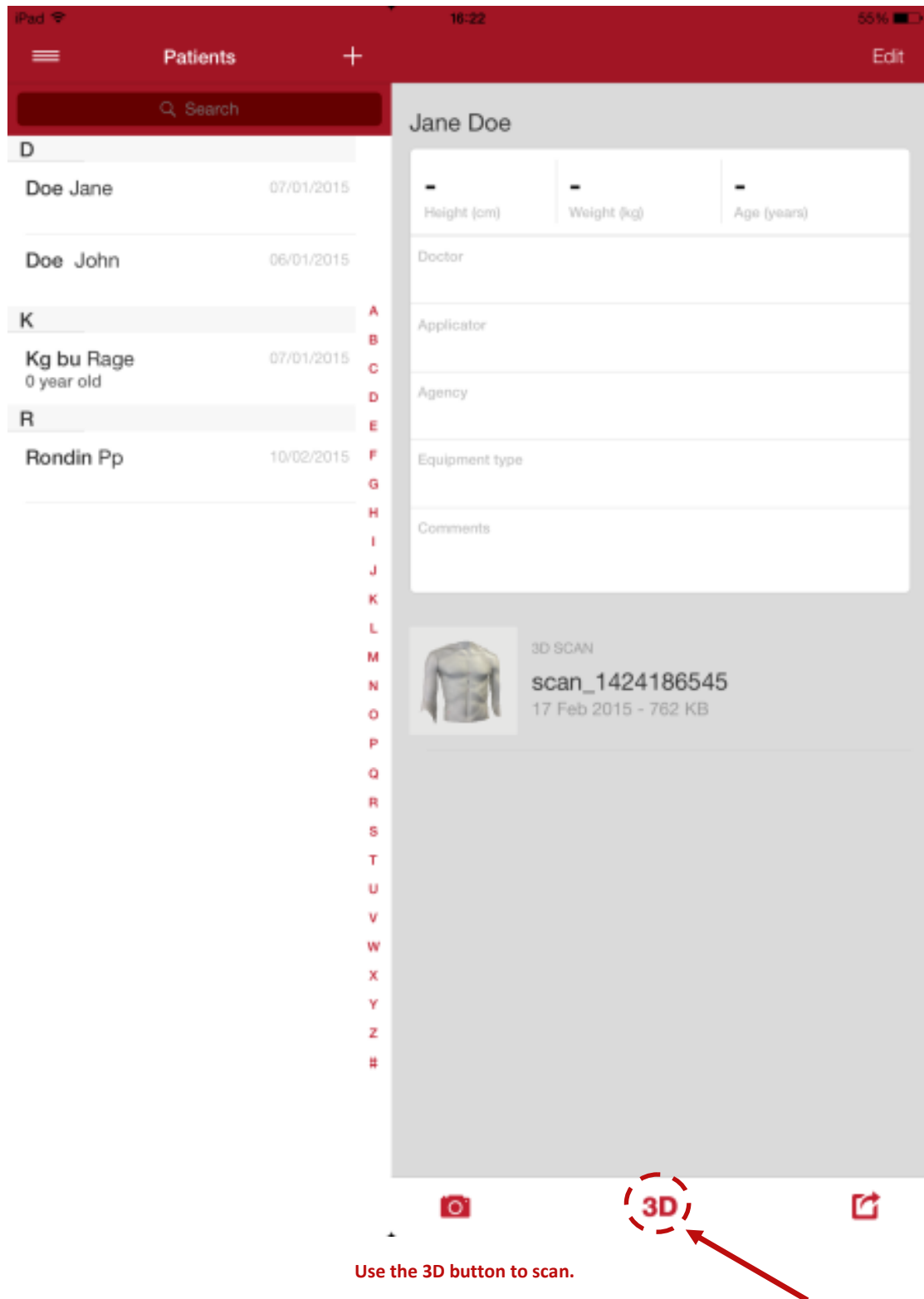
To scan your patient, you must use the “3D” button.


Select the part you want to scan by extending or shrinking the box. The area you will scan will appear in red.

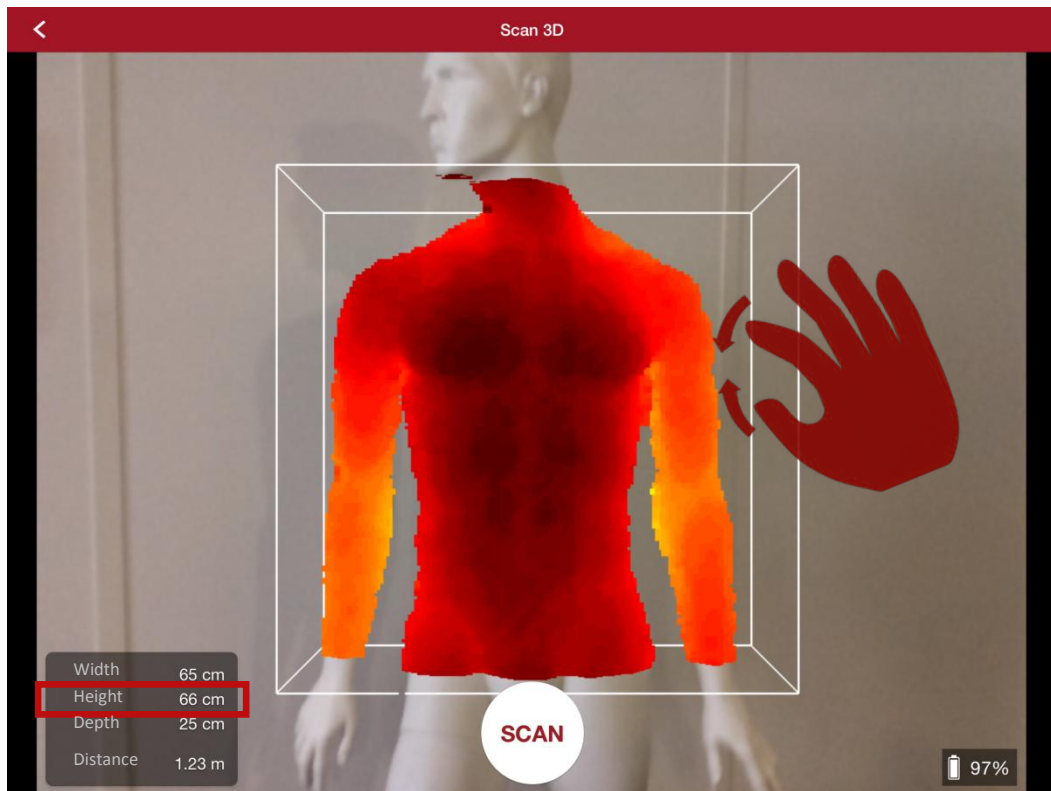To modify the size and the position of the box use the gesture:

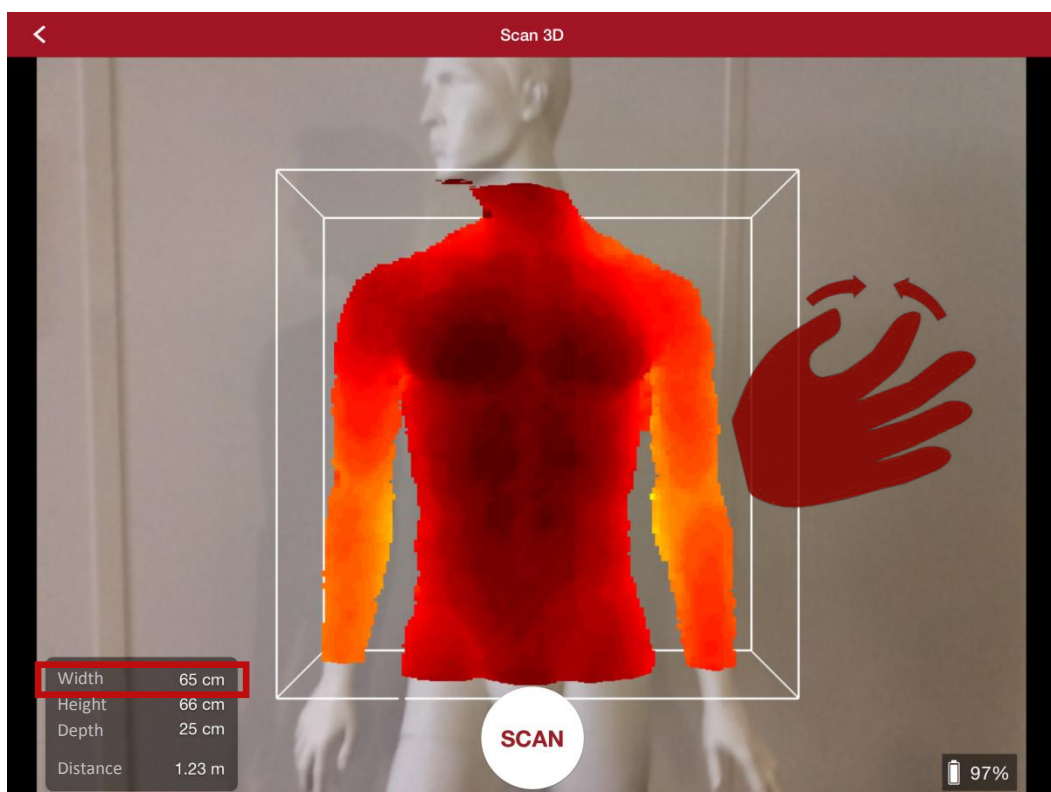
Zoom in-Zoom out vertically to modify the height of the box

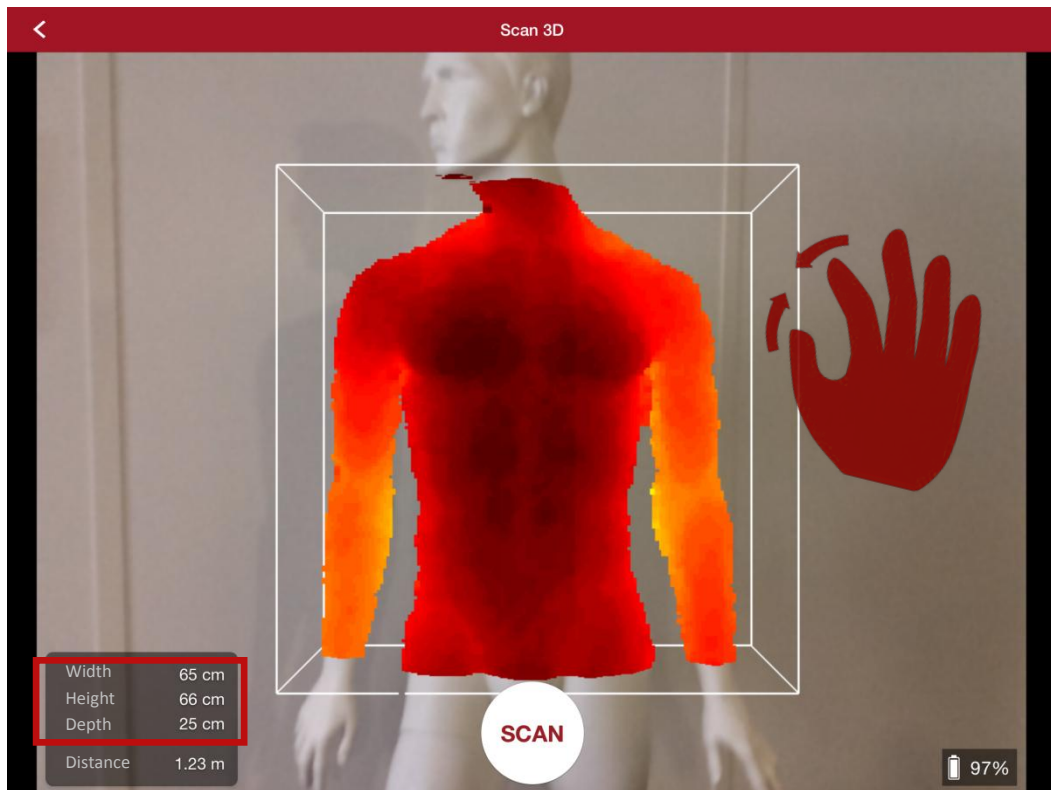
Zoom in-Zoom out horizontally to modify the width of the box

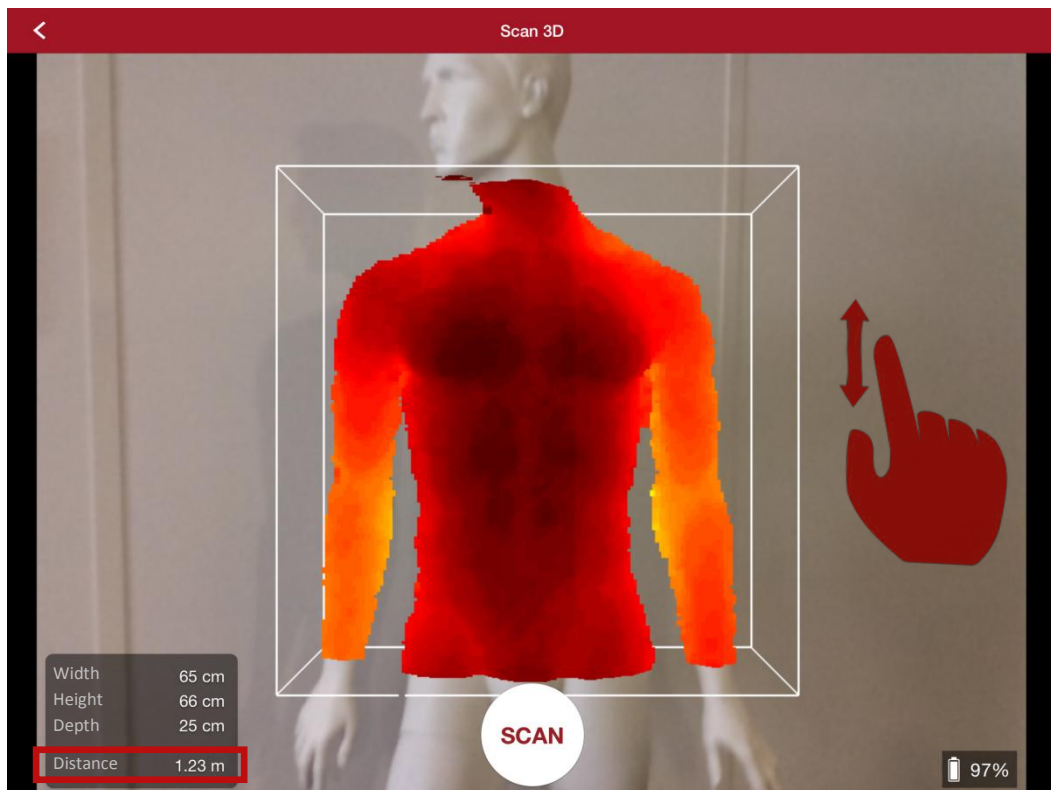
Zoom in-Zoom out diagonally to modify the three dimensions of the box

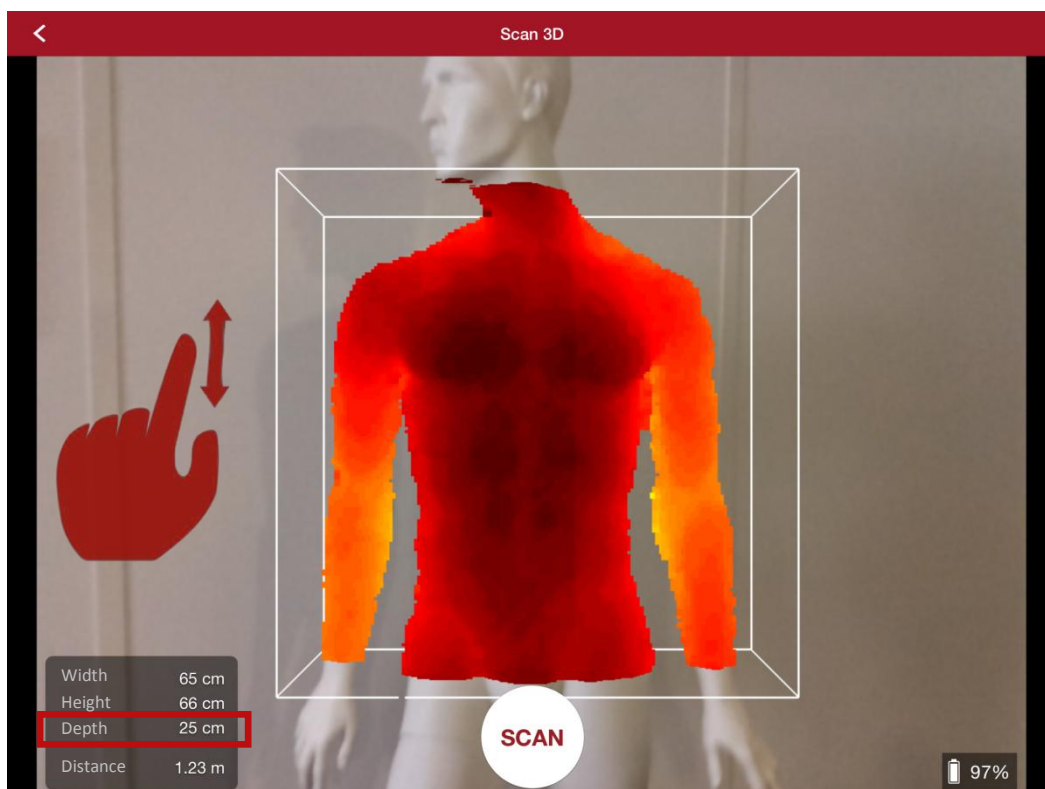
Swipe down-Swipe up on the right of your screen to modify the position of the box (distance between the sensor and the middle of the box)

Swipe down-Swipe up on the left of your screen to modify the depth of the box (the distance is also modified as it changes the position of the center of the box)

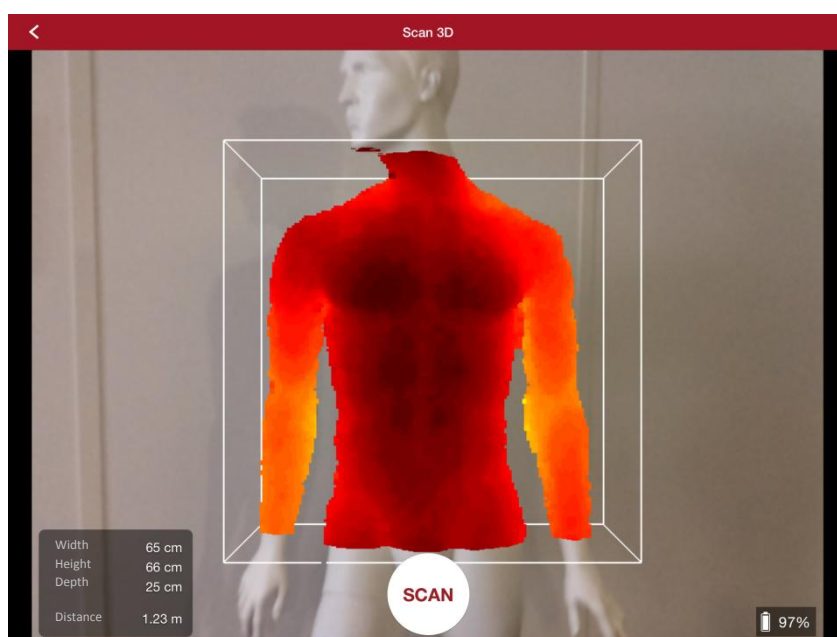
Advise: Watch the box size and position on the bottom left corner of your screen.

The red part will be scanned

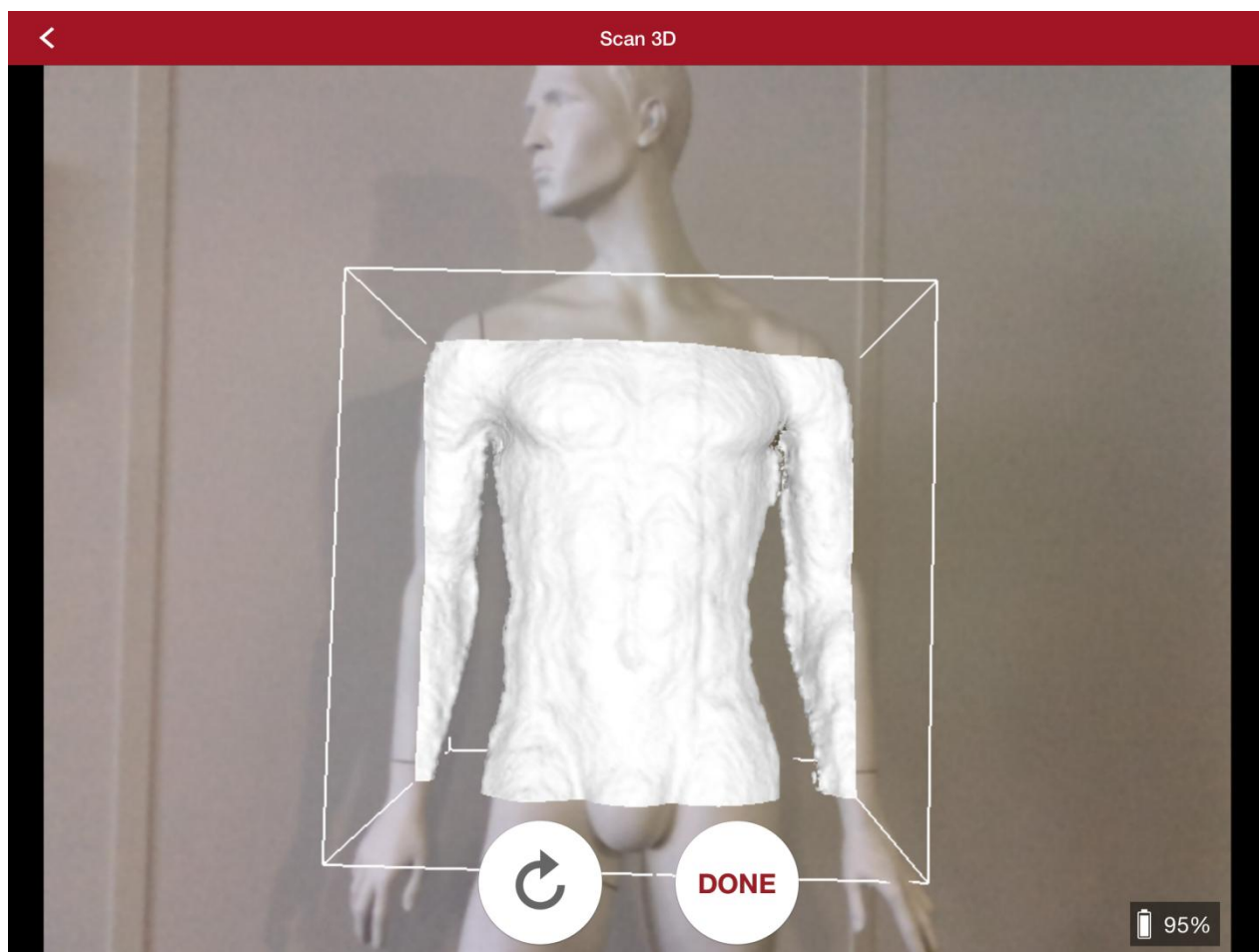Click on “scan” when the selected area suits, and begin scanning. Everything that was not in red before clicking on the “scan” button won’t be scanned.

Begin the scan

Turn around your patient to scan him. Come closer to the patient for a better precision, without going nearer than 40cm, which is the structure sensor limit to scan.

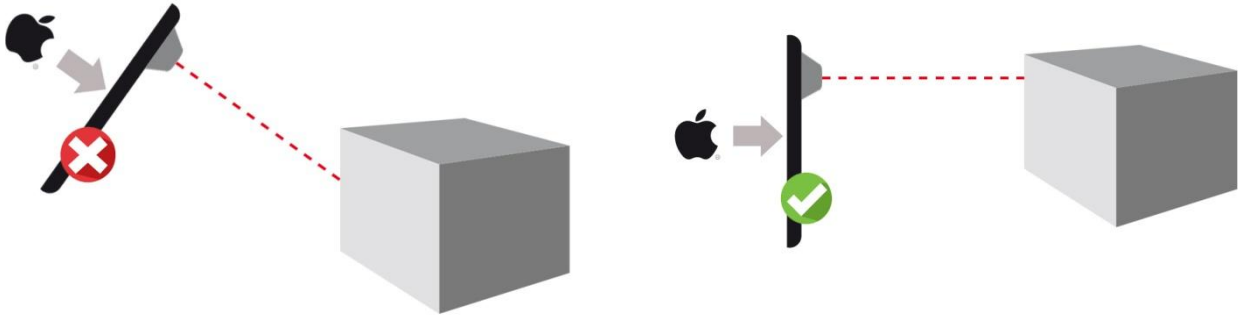
Advise: The smaller the box, the better the precision.

The precision of your 3D model will depend on the size of the initial box. The smaller it is, the better the precision of your measurements. We advise you to make the box the smaller you can, depending on the orthopaedic device you want to make.

The structure sensor projects infrared light on the surface. To obtain better results:

- Position the structure sensor the more perpendicularly possible to the model
- Scan from short distance (40cm)

Advise: To optimize the result of the scan, position the Structure Sensor perpendicularly to the model.

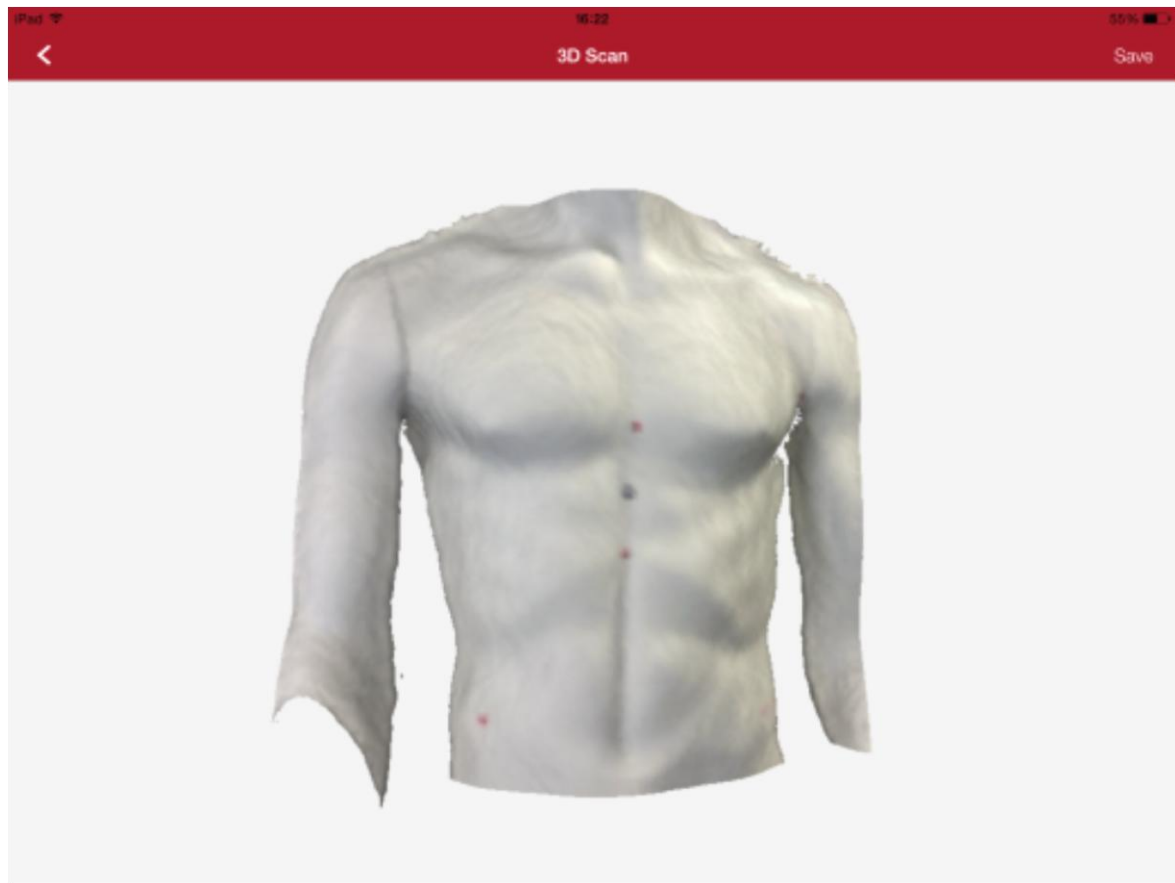
Advise: To optimize the result of the scan, position the Structure Sensor at 40cm from the model.

Click on “Done” to end the scan.

The return button allows you to begin again your scan from the box setting step.


See the result of your scan

Send you files

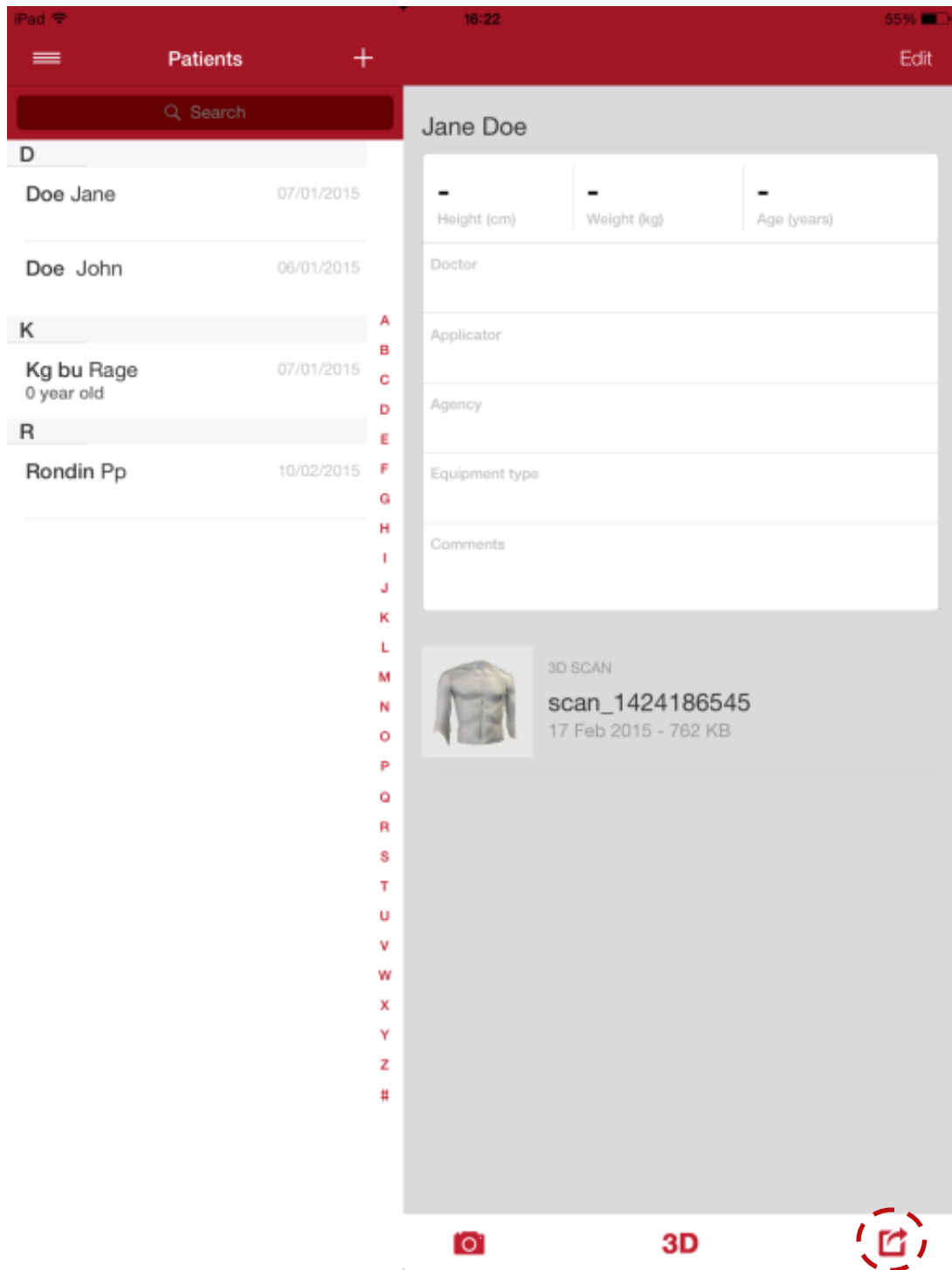
Send your scan by email or dropbox if necessary.

Send your scan

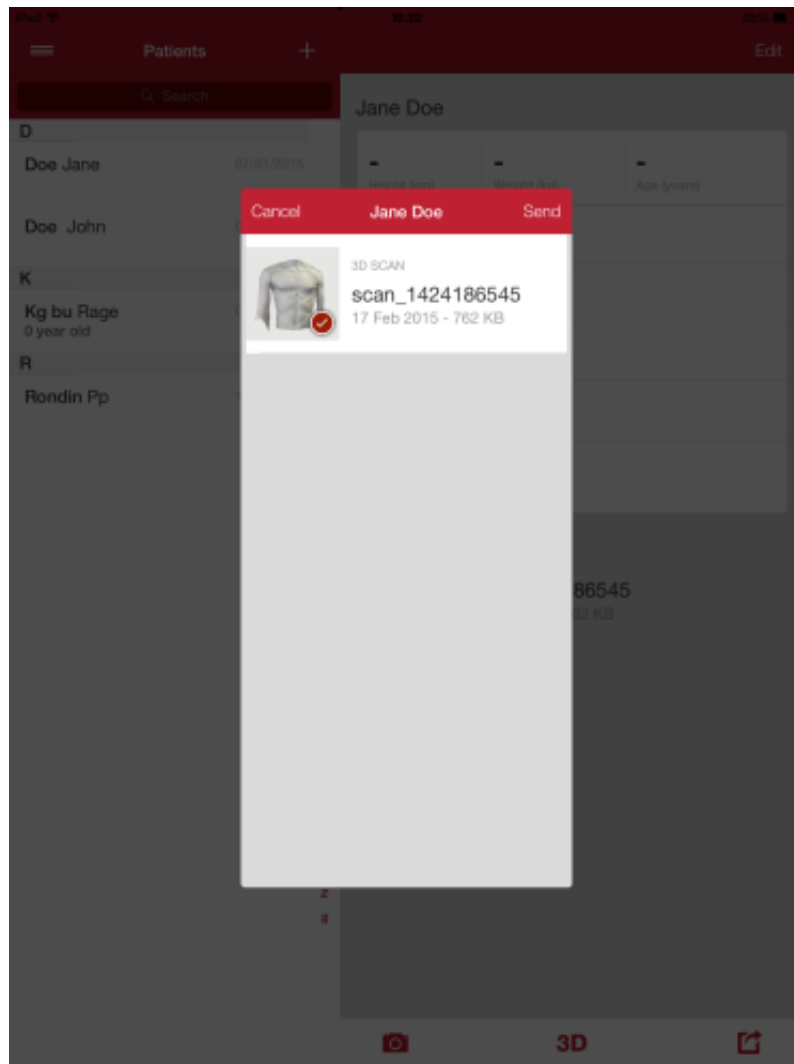Select the files you want to include in your captevia file.

Select the files

Open the file in **RODIN4D neo**
GLOBAL CAD/CAM FOR P&O

Download the captevia file and double click on the icon.

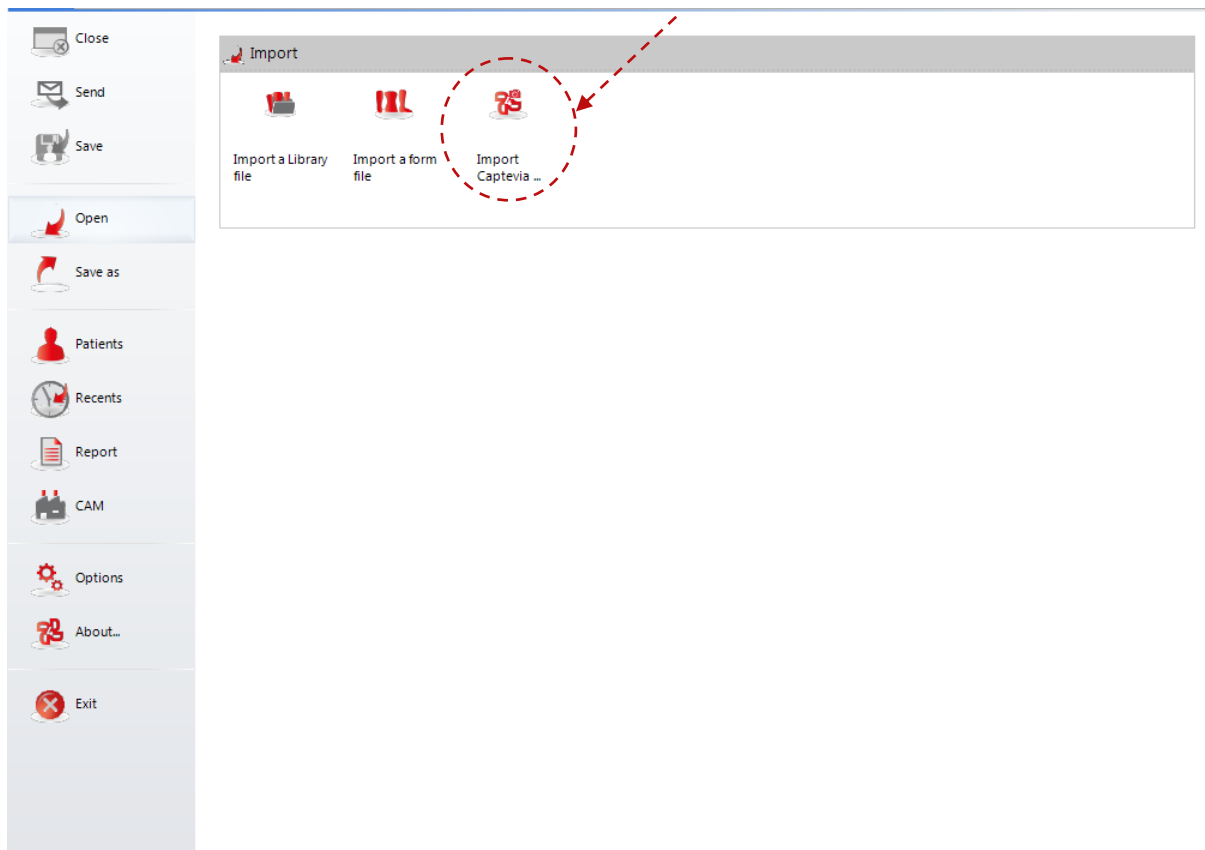
Double click on the file

Rodin4D Neo opens up and shows you the content of your file.

You can also open the Captevia file from Rodin Neo interface (Files ->Open menu) since version 10.0.85.

Open your Captevia file directly from Rodin Neo

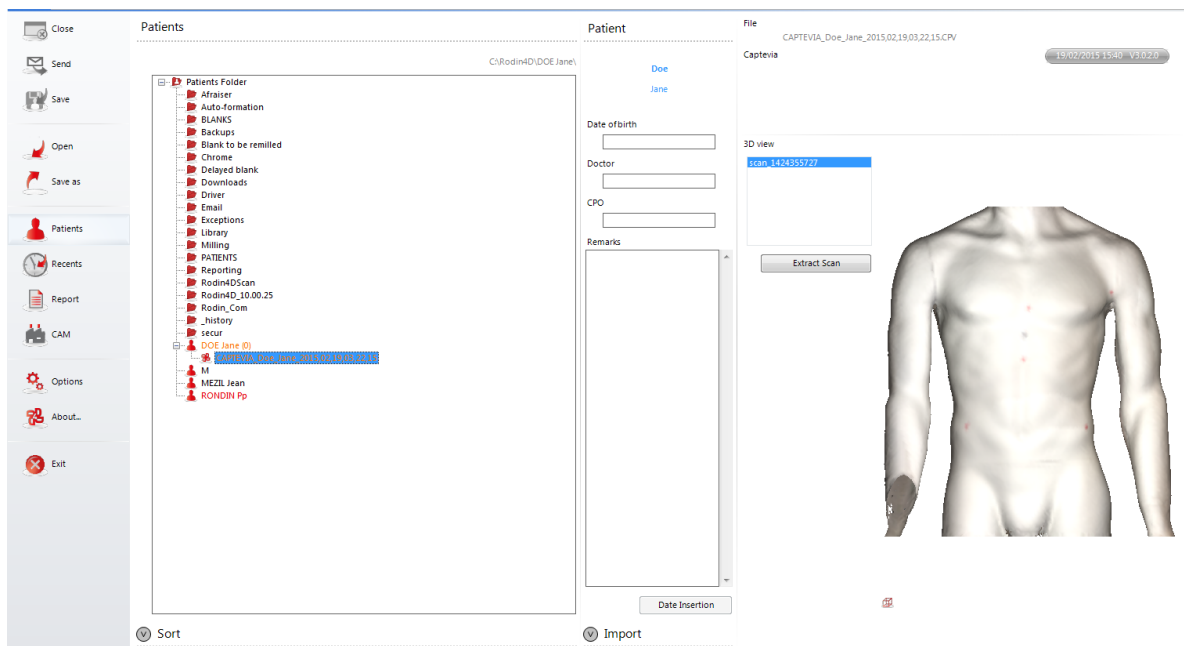
You can either create a new patient or copy the captevia file in an existing patient folder. Once you have chosen, close the window and find your patient in the patient manager.

Display the content of your captevia file


From your patient manager, you can extract the data of your captevia file. Select the desired scan in the 3DView list, and use the button “Extract Scan”.

Visualize the file in your patient manager

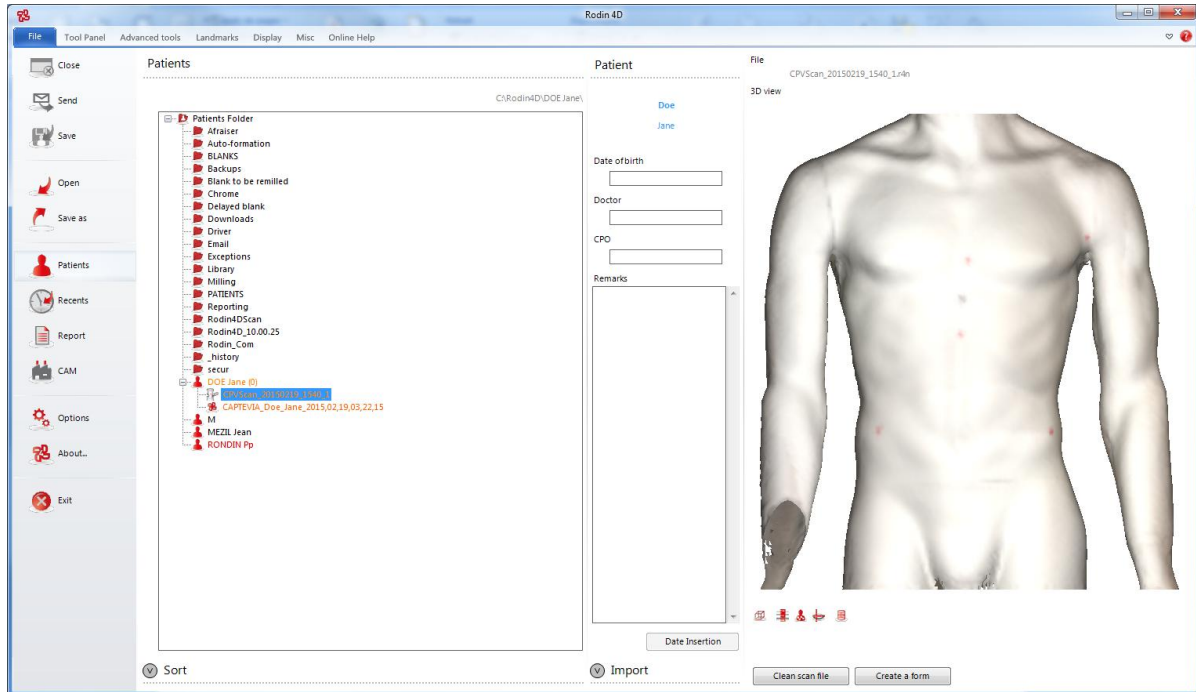The scan appears in the patient folder. You can either create the Rodin form directly with the “Create form” button or clean it up first. To do so, click on “clean scan file”.


Extract the scan in your patient folder

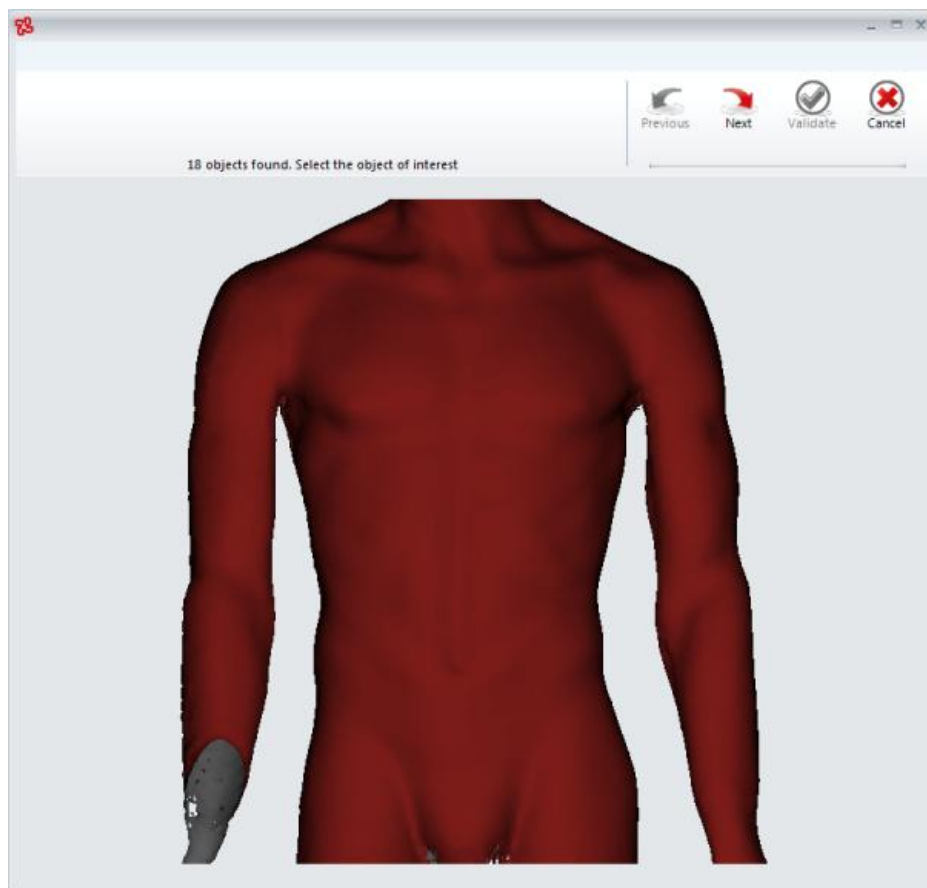
A cleaning window appears. Follow the steps and click on validate when you are finished.

Follow the steps of the cleaning assistant.

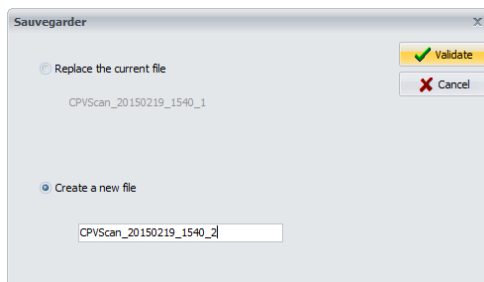
You can either replace the current file or create a new file. To do so, you must choose the name of your new file.

To create a new file, choose the name of your new file

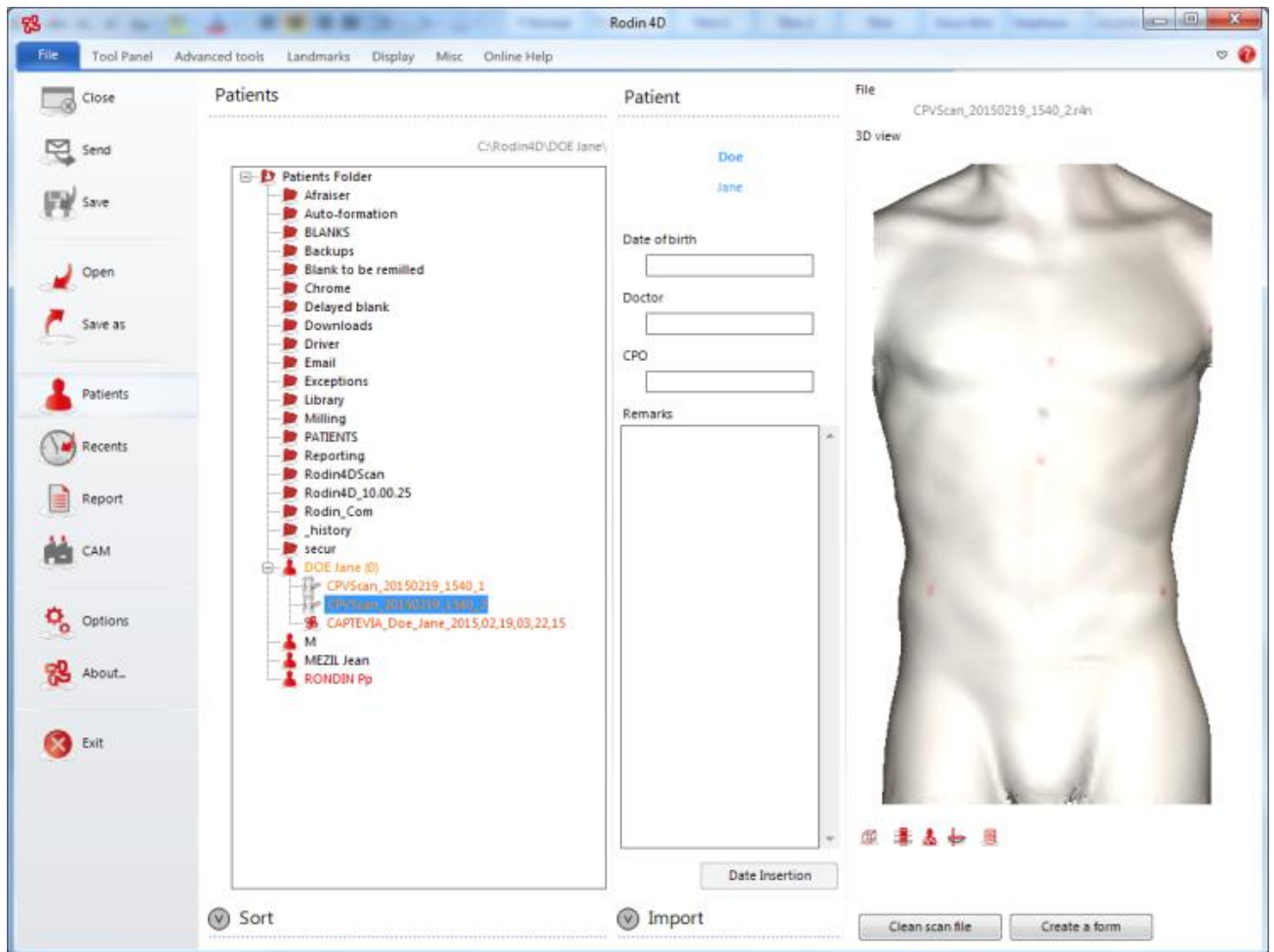Once you have cleaned your scan, you can create your Rodin form with the button “Create form”.

Create the Rodin form

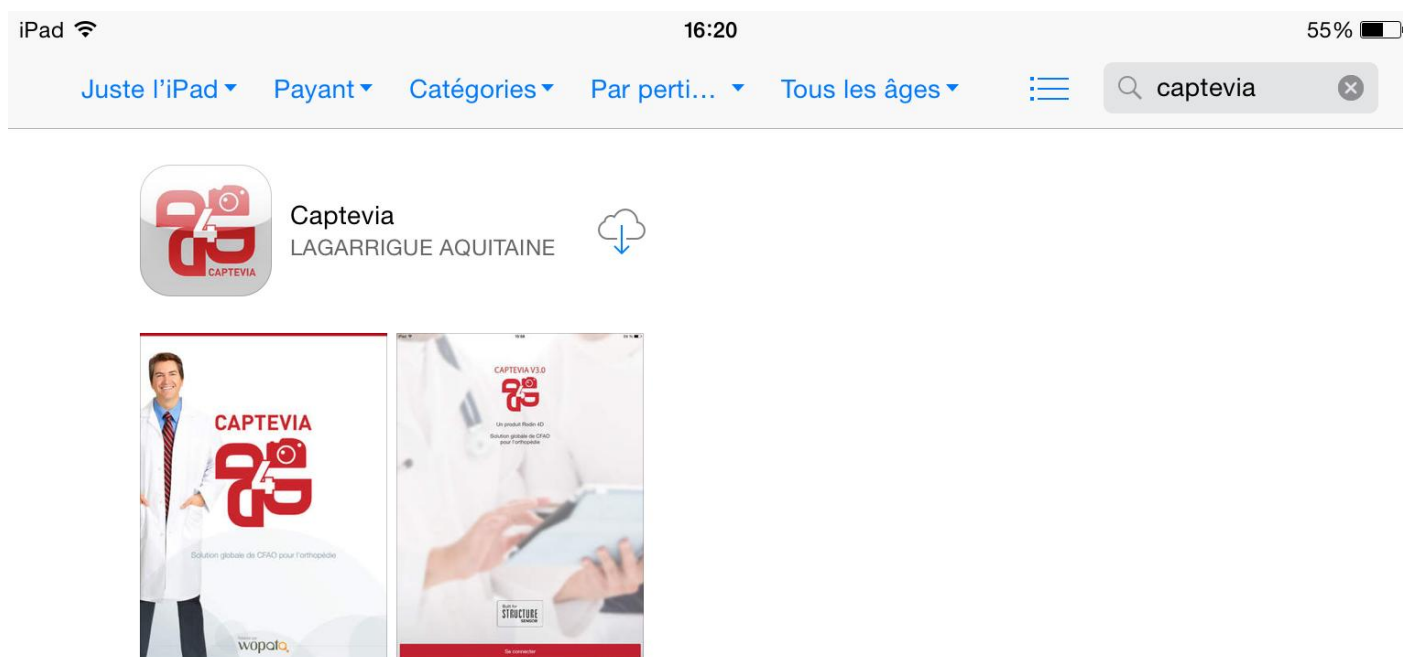
You are now ready to design your orthopaedic device!

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The screenshot shows the 'Add a patient' form in the Captevia app. The form is overlaid on a 'Patients' list. The form fields include: SURNAME, FIRST NAME, DATE OF BIRTH, HEIGHT (cm), WEIGHT (kg), DOCTOR, APPLICATOR, AGENCY, EQUIPMENT TYPE, and COMMENTS. The 'Add a patient' button is highlighted in red. The background shows a list of patients with names like 'Doe Jane', 'Doe John', 'Kg bu Rage', and 'Rondin Pp'.

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The screenshot shows the 'Add a patient' form in the Captevia app. The form is overlaid on a 'Patients' list. The form fields include: SURNAME, FIRST NAME, DATE OF BIRTH, HEIGHT (cm), WEIGHT (kg), DOCTOR, APPLICATOR, AGENCY, EQUIPMENT TYPE, and COMMENTS. The 'Add a patient' dialog has 'Cancel', 'Add a patient', and 'Save' buttons at the top. The background shows a list of patients: Doe Jane, Doe John, Kg bu Rage 0 year old, and Rondin Pp. A keyboard is visible at the bottom.

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